

# Content

01. Important Safeguards.....	3
02. Contraindications .....	4
03. Connecting Accessories and Start-Up .....	4
Treatment selection .....	7
DEPILATION PROGRAMS .....	8
How to adjust device settings .....	9
IONTOPHORESIS PHASE .....	11
05. System Screen Configuration .....	13
06. Consultation before Treatments .....	14
07. Hair Growth Cycles .....	15
09. Client “Post Care Instructions” .....	16
10. Accessories for Evolution 7HD .....	17
11. Important .....	17
12. Warranty .....	18
13. Specifications.....	18

All rights reserved. No part of this work covered by the copyright hereon may be reproduced or used in any form or by any means– graphic, electronic, or mechanical, including photocopying, recording, taping, or information storage and retrieval systems– without permission of the company.

© Silhouet-Tone Corporation 2021



***We appreciate your confidence in Silhouet-Tone's products and value your decision to purchase the Evolution 7HD.***

The purpose of this manual is to familiarize the operator with the features, operation and maintenance of the unit.

The new Evolution 7HD is an innovative permanent hair removal system with a communication interface that enables taking advantage of any future software updates. This will guarantee, your electrolysis unit will always remain on the leading edge of technology.

### **VECTOR MODULATION™ (Exclusive Technology)**

After years of research, Silhouet-Tone and its team of engineers have conceived the **Vector Modulation™ technology**, an advanced current delivery system. At its core, **Vector Modulation™** modulates the current in both intensity and duration while the needle is in the follicle. This major breakthrough in permanent hair removal, firmly places electrolysis at the forefront of this industry by solving three important issues:

1. **Pain: is reducing the pain to the lowest level possible.**
2. **Efficiency: is maximizing treatment result.**
3. **Satisfaction: achieves permanent results faster.**

**Sequential Modulation™** is more accurate and stable than any other existing technology. This technology uses a two millisecond segment current, the fastest on the market, to maximize the transfer of energy through the follicle.

Now, with the introduction of the Evolution unit, we push further the innovation and we create the **Vector Modulation™**. This technology emits complex 7HD wave forms to deliver greater amounts of energy in the hair follicle associated by unprecedented and optimal comfort. Fractional modulated current in 2 millisecond segments eliminates discomfort caused by current pulsation or repeated application. Energy remains constant at the follicle base, optimizing the destruction of hair growing cells

### **Benefits**

- Uses the only technology to remove hair permanently.
- Permanent rapid destruction of all "hair growing cells".
- Achieves quicker permanency.
- Is effective on all hair types (fine, medium, coarse hair and very coarse hair).
- Eliminates ingrown hairs.
- Does not damage the surrounding tissue.
- Does not leave hyperpigmentation or scars.
- Specific programs for curly hair

# 01. Important Safeguards

## Read the following instructions before using this apparatus!

1. This apparatus is to be used only for purposes that are approved and described in the following manual. Use only with attachments recommended by the manufacturer.
2. Prior to each use of the system, inspect all cables and attachments for any damage, excessive wear, or crimping that could affect system performance or safe operation. Never use the system if any damage is visible.
3. Never perform a procedure on inappropriate or sensitive areas.
4. Do not use this apparatus if the power cord has been damaged, if it is not working / operating correctly or damaged. Return apparatus to service center for verification or repair.
5. Keep power cord away from heat source.
6. The power cord should be routed so that they are less likely to be walked on or pinched by items placed on or against them. Pay particular attention to cords at plugs, convenience receptacles, and the point where they exit the unit.
7. **WARNING:** To avoid the risk of electric shock, this equipment must only be connected to a supply main with protective earth.
8. **WARNING:** Caution should be exercised to avoid spilling any liquids on the unit.
9. Never immerse the accessories in any type of solution.
10. Never pull the power cord to disconnect the equipment from the mains. Always do so by firmly grasping and pulling the connector. Always allow enough space between the equipment and the mains for easy disconnect or plug in access.
11. This apparatus should only be operated by an appropriately trained and certified personnel possessing knowledge of the body physiology and only in a professional setting. If you have any questions regarding the operation of this unit, contact Silhouet-Tone or your local distributor.
12. **Grounding instructions:**

This unit must be properly grounded. In the event of a short-circuit, the power cord's ground prong will reduce the risk of electric shock by providing an escape route for the current.

The power cord is equipped with a grounding wire and three-pronged plug. The power cord must be plugged into a properly installed three-pronged grounded outlet.

**DANGER: THE IMPROPER USE OF A GROUNDING PLUG MAY CAUSE ELECTRIC SHOCK!**
13. Do not use any adaptors with this unit. Do not modify the plug if it does not fit the outlet. Have an electrician install a suitable outlet.
14. Do not overload wall outlets and extension cord; this can lead to an increase risk of fire or electric shock.
15. **All maintenance work** must be done by SILHOUET-TONE or an authorized service representative.

**Attention: Electric shock hazard. Do not attempt to service this unit yourself. Opening the unit may expose you to high voltage or other hazards, and will void your warranty.**

**Refer all servicing to qualified service personnel.**
16. To insure continuous protection against fire, always replace the fuse with one of identical type and amperage, as approved by the manufacturer.
17. Slots and openings in the unit casing are for ventilation. They are there to ensure reliable operation and prevent overheating. These openings must not be blocked or covered. This unit should not be placed in a built-in installation unless proper ventilation is provided for.
18. Always have a good contact between the contact electrode and the patient's skin.
19. Always have a good contact between the patient applied part and the skin.
20. Never immerse the needle holder in any type of solution.
21. It is imperative that the needle be changed, if at any time the needle holder is dropped.
22. It is imperative to carefully inspect the needle holder for damage if it is dropped.
23. Never connect any foreign accessory into the unit other than the appropriate needle holder, contact electrode and foot pedal switch.
24. Never connect the needle holder, contact electrode or foot pedal switch into any foreign unit.

**Save these instructions.**

## 02. Contraindications

**PRIOR TO TREATMENT, EACH CLIENT MUST FILL OUT CLIENT RECORD CARD AND THE CONSENT FORM.**

### CONTRAINDICATIONS:


- Pacemaker.
- Metallic implant.
- Insulin-dependent or severe diabetes.
- Pregnancy.
- Prescription anti-coagulant (blood thinning drug) as Warfarin, Coumadin or antecedent embolism or phlebitis.
- Viral hepatitis.
- HIV.
- Cancer.
- Keloid scars.
- Rosacea (during active phase or with severe acne).
- Infectious acne.
- Herpes simplex (cold sore).
- Any health problems or medical condition that can delay healing process.

**⚠ IMPORTANT: When in doubt, ALWAYS refer client to physician or ask for a written physician's authorization.**

## 03. Connecting Accessories and Start-Up

**⚠ CAUTION: Risk of electrical shock, do not remove back cover. Refer servicing to qualified service personnel.**


**Back side:**

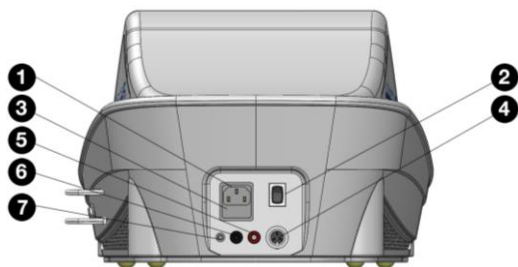

1. Power outlet.
2. ON / OFF switch.
3. Fuse box.
4. Pedal outlet (big black outlet).
5. Soothing roller electrode (red outlet).
6. Contact electrode outlet (small black outlet).
7. Satellite 2 LEDs red & green (silver outlet).

**Front side:**

## 8. Probe Holder Connection.

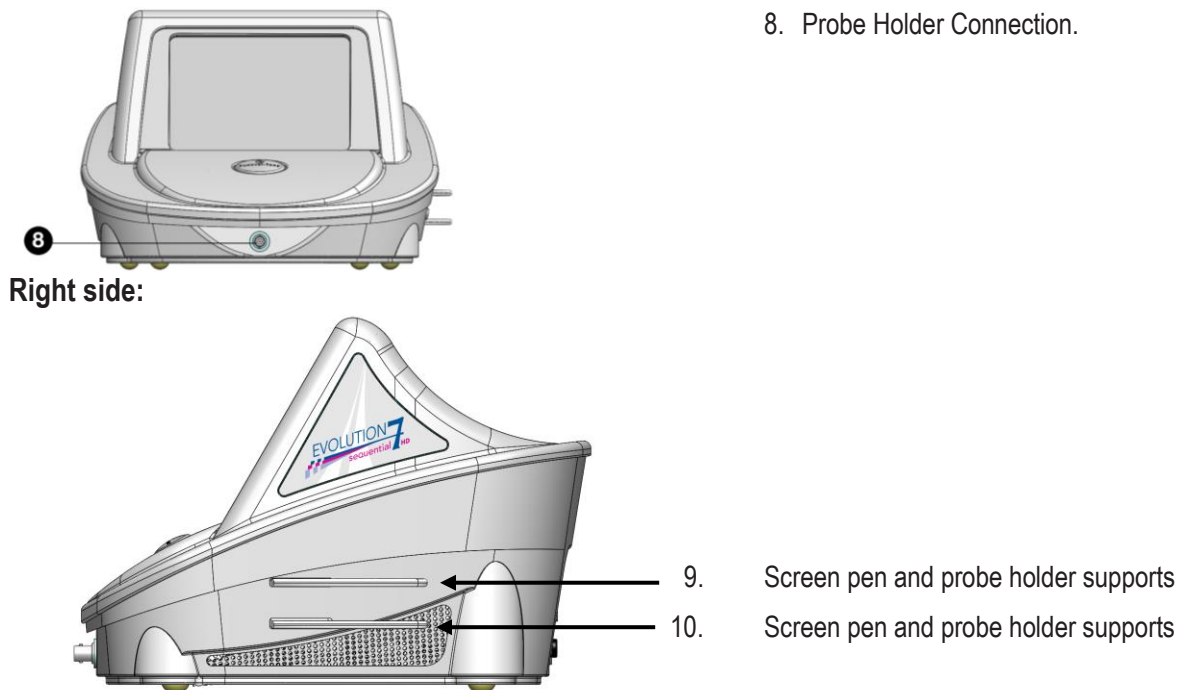

### Connections and accessories:



**NOTE:** Make sure that the ON / OFF switch is in the OFF position before connecting any accessory.

#### Back panel:

- **Power cable** to be connected into # 1 Power outlet.
- **Pedal** to be connected into # 4 Pedal outlet (black big outlet) on the rear unit's side.
- **Red extension cable** to be connected into # 5 Roller electrode outlet (red outlet) on the rear unit's side while other end inserted into the **Soothing roller** outlet.
- **Black extension cable** to be connected into # 6 Contact electrode outlet (black small outlet) on the rear unit's side, while other end is inserted into the **Electrode** outlet.
- **Satellite** to be plugged into # 7 Satellite outlet (silver outlet) on the rear unit's side.


#### Front panel:

- **Cable for Probe holder:** thicker end to be connected into # 8 Probe Holder Connection on the front unit's side (insert and rotate to insure holding protection) while other end inserted into the **Probe holder**.

Now you can Turn **ON** the unit.


### Turning the unit ON


1. To turn the unit on, use the power switch located at the back of the unit to the "ON" position (1).
2. The screen will display a **green little guy + the mention Android**
3. This screen will appear the first time you turn on the machine (or after you perform an update) :

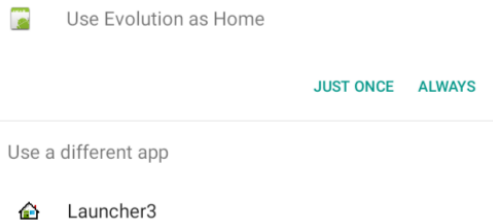

4. Press **“Use Evolution as Home”**. Next screen will appear :
5. Then press **Evolution**.
6. The unit will take few minutes to load all programs.
7. It is ready to use when the screen displays the main menu **“Depilation”, and “Iontophoresis”**



8. **To access to the Instruction Manual press “Icon Instruction”** button at the bottom of the front screen.
9.  The **“TOOL key”** on the bottom left of the screen enables to access system's setting parameters: screen brightness, beep, insertion delay, language, updates, and Wi-Fi configuration (see section 6).
10. To change the language select desired language and it will be kept in memory for following uses.

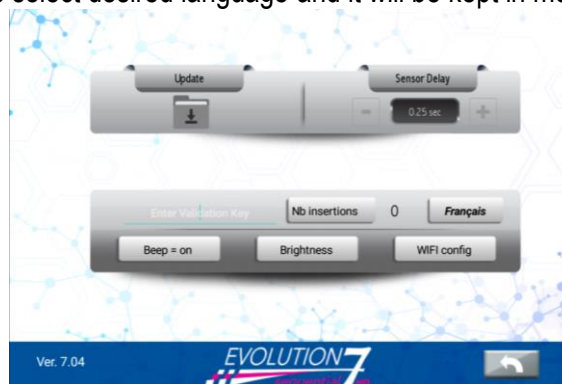

 Use the return icon to go back at the previous screen.

To turn the unit OFF use the power button located at the back.

## 04. Treatment Screen

**! IMPORTANT:** It is highly recommended to carefully read and understand the instructions manual to properly use your electrolysis system by touching “Instructions” button in the bottom of the screen.

**! CAUTIONS:** The touch screen interface is very sensitive, therefore:


- ✓ **ALWAYS** use the screen pen provided with the unit to touch the screen. No need to press hard.
- ✓ **NEVER** use sharp objects as these may damage the screen.
- ✓ **NEVER** put the probe holder or the tip of the needle in contact with the screen surface. The probe carries high voltage and will damage the screen.

**! CAUTIONS:** Never proceed to use the unit if the display screen is inoperative.

**! WARNING:** Never touch the screen with the probe.

## Treatment selection


Use the main menu to select Depilation or Iontophoresis.

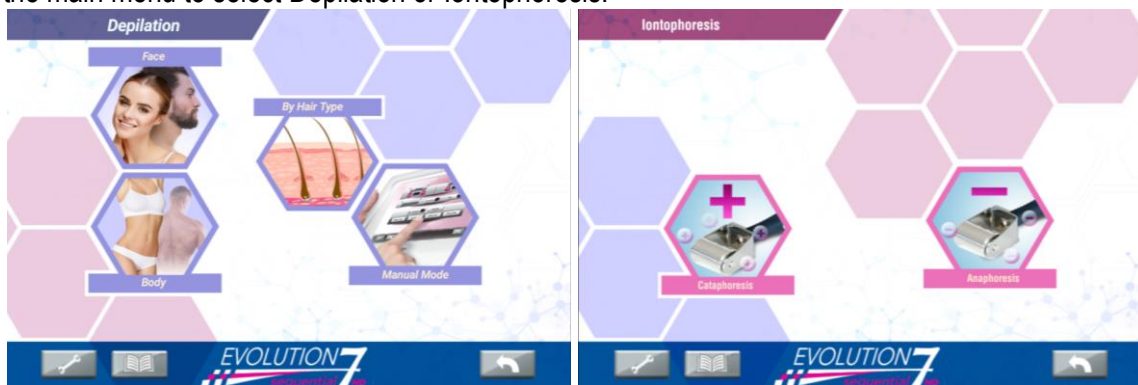


 The **Back key** will bring up the previous screen.

4 Selection buttons are available in the Depilation menu: Face, Body, By Hair Type and manual mode:

- Face or Body icon brings up face or body areas to be treated. Press desired area to select treatment.

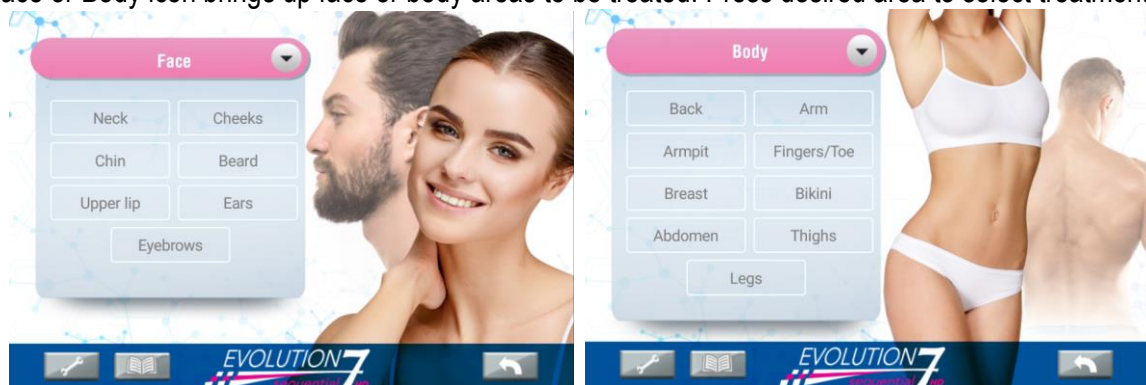

- The Manual Mode icon brings the manual treatment settings.
- The By Hair type icon selects generic intensity settings based on hair size.

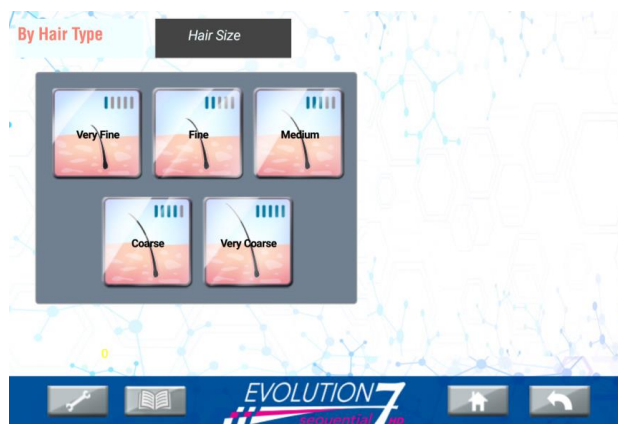


**Note:** before any treatment verify if all jewellerys are removed.

## DEPILATION PROGRAMS

### 1. Face and Body Programs

Each area to be treated has specific waveforms and modulation programs designed for its own sensitivity and degree of difficulty. A fine hair on the chin does not respond the same way as a fine hair on the upper lip. These programs insure maximum comfort. When selecting treatment area, the unit displays the hair types. Choose hair type by pressing the corresponding icon: **Very Fine**, **Fine**, **Medium**, **Coarse**, **Very coarse**.

The example below shows following selection sequence: Depilation / Face / Chin / Very coarse / Blend;

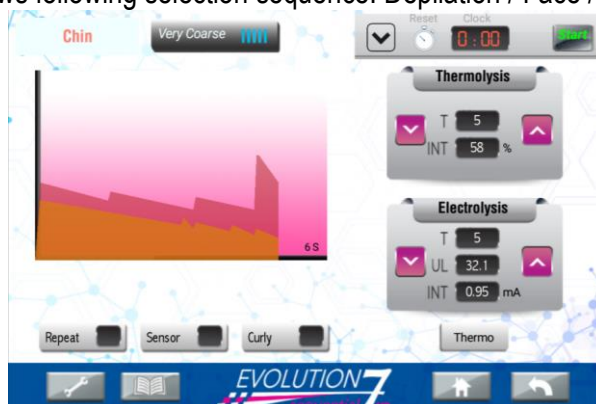

To select a different hair type while staying in the same area, simply press the hair icon.

When treatment parameters are displayed on the screen, you're ready to start the treatment.

Blend mode (high-frequency energy combined with direct current) is available for all hair removal options in preprogrammed mode. You can select it by activating the "Blend" key and deactivate it to return to "Thermolysis" mode by activating the "Thermo" key.



We advise you to adjust (increase and decrease) the pre-recorded parameters (High-Frequency and direct current) in the "Blend" mode. Select this mode according to your own professionalism and judgment, knowing that it is intended to treat coarse, deformed, persistent hair deeply implanted in small areas.

On the treatment screen, the graph systematically represents the shape of the wave produced by the device. It is specific to each zone, each technique and each size of hair.

## 2. Hair type programs

Same settings as in Face and Body programs, only the treatment zone does not appear.

## 3. Manual Mode


You have the possibility of working by choice of hair removal technique (without hair gauge or treatment area) and you let the parameters change according to your free interpretation:

- In Thermolysis mode, the parameters are zero. Select the **Thermo** key.
- In Electrolysis mode, the parameters indicate by default a time for the wave to pass. Select the **Elec** key.
- In Blend mode, a basic parameter is proposed in order to start the treatment freely. Select the **Blend** key.

How to adjust device settings

### IN BLEND MODE

- The arrows in Electrolysis are used to increase or decrease at the same time the UL and the intensity. Changing treatment parameters will affect potency.
- Time is not editable in Blend mode for High Frequency current (Thermolysis window). It was calculated judiciously to produce the desired reaction of heating water molecules to facilitate the production of lye.
- The arrows in Thermolysis are used to increase or decrease the intensity in %. Changing treatment parameters will affect potency.
- Time is not editable in Blend mode for galvanic current. It was calculated judiciously in order to produce the desired reaction of sodium hydroxide (lye) production.

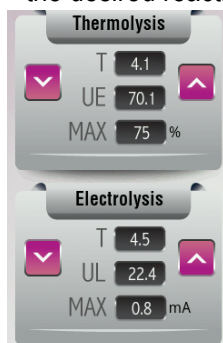

T=Time/seconds represents the duration of Thermolysis/Electrolysis wave.


INT = Indicates the intensity of the HF energy wave

UL = Total units of lye produced.

MAX= The maximum value that DC current will reach.

### IN THERMOLYSIS MODE

- Arrows in thermolysis are used to raise or lower the intensity in %.
- Time can be modified to gain in efficacy and personalise treatment.

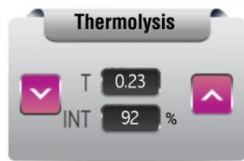

T= time in seconds: indicates the duration of the themolysis wave

INT = Indicates the intensity of the HF energy wave



Time is the treatment time

Other treatment parameters:




**Repeat** key can be used to repeat the modulation wave up to 2 times without re-inserting the needle or having to activate the pedal. Pressing this key will increment the repeat count to 1. Pressing once again will reset it back to zero. Zero means that the modulation wave will discharge once. A "Repeat 1" will discharge once and then repeat.



**Sensor** key activates the **Insertion Delay**. Blue indicates that the Insertion delay function is on. When insertion delay is activated, current automatically flows upon detecting insertion. When Sensor is Off (black), current must be activated with the pedal.



**Curly** key provides extra bursts of energy when working with difficult or distorted hair follicles. Blue indicates that function is ON, black indicates it is OFF.

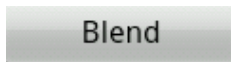

**Blend** key allows to switch from Thermolysis to Blend mode. When blend is selected, an extra Electrolysis dial appears. It can be used to modify galvanic parameters. Pressing the Thermo key will switch back to Thermolysis mode.



The hair selection bar displays the selected hair. The blue lines represent a graphic of the power output. To select a different hair size while working in the same area, simply press this bar key, the hair selection icons will appear.



Press start to activate clock. Press Reset to reset the time to 0. Press the arrow to change from Clock to Chronometer.



Press + or minus to select desired time and press start to start the Chronometer. Press the arrow to change from Chronometer to Clock.

The BOOK icon allows you to access the complete instruction manual during your treatment if necessary.

The home icon at the bottom of the Depilation treatment, allows going back to the start menu in order to access to the next treatment, Iontophoresis phase.

### If the hair resists

Several steps to take successively (not all, but follow the recommended order)

- Use of the basic criteria configured in the device
- Increase in intensity between 2 and 4 points
- Increased HF current passage time
- Change in hair size (switch needle if necessary)
- Needle size change (larger size)
- Select curly mode (according to area)

- Double the pulse (repetition) by programming the device or using pedal (adjusting the technique of applying the needle in the follicle)
- Blend mode selection and needle adjustment accordingly (depending on hair size > from medium).

### The hair sticks to the needle

Do the opposite of the description above > the intensity is too strong.

**⚠ CAUTION:** the definitive electric hair removal device Evolution 7HD is operator dependent and training dependent. Its use is subject to specific vocational training at the Silhouet-Tone training centres.

Silhouet-Tone declines any responsibility if the basic training in permanent electrical hair removal is not followed by the technician.

## IONTOPHORESIS PHASE

Select the Iontophoresis key on the main menu, you can then select **Cataphoresis** or **Anaphoresis**.



**Note:** The polarity of the DC current is pre-set, inside the unit, by a relay. Therefore, there is **no need to modify** connection of the electrodes when switching between **Cata** and **Ana**.

When you select **Cata**, roller's electrode (active electrode) is automatically loaded with a positive charge (+). When you select **Ana**, roller's electrode is automatically loaded with a negative charge (-). Default treatment time is 3 minutes. DC current intensity can be increased or decreased by 1/10<sup>th</sup> of a mA (milliamp) increment. Pressing the **On** key starts treatment. Reaches the pre-set time. To start a new treatment exit this screen or increase the set time. **The unit will not allow the time to restart if the limit has been reached.**



### Before treatment make sure that:

- All jewelry must be removed.
- Place the contact electrode covered with wet gauze, under the shoulder, when working around face area or upper body or under the thighs when working on lower body.

**CAUTION :**

- **NEVER** use the roller directly on the skin (without gel).
- **For roller maintenance clean all gel residues on the roller after each use.**
- **Use bactericidal wipes to disinfect the roller.**

**(-) ANA**

Ana is applied **BEFORE** depilation treatment to open hair follicles. **NEVER** use Ana after depilation treatment.

- Apply a layer of Sensibilis Lotion (pre-depilation lotion).
- Use the roller electrode over the treated area for 3 minutes. (Never use metallic roller on bare skin).
- Remove gel and rinse the skin.
- To prevent electrode deterioration, remove gel residues from roller electrode.
- Use bactericidal wipes to disinfect roller.

Treatment should be comfortable and soothing. Client should only feel slight tingling. Lower intensity if client is uncomfortable. A tingling may also be felt under the contact electrode (make sure it is covered with wet gauze).

Recommended intensity settings for “Ana”(Before electrolysis):

- **Face:** 0.30 mA to 0,80 mA
- **Body:** 1.00 mA to 1,50mA

**(+) CATA**

Cata is applied **AFTER** each depilation treatment to sooth the skin.

Treatment procedure for Cataphoresis:

- Apply a layer of Sensibilis Gel (post-depilation soothing gel).
- Use the roller electrode over the treated area for at least 3 minutes. (Never use metallic roller on bare skin).
- Remove gel and rinse the skin.
- To prevent electrode deterioration, remove gel residue from roller electrode.
- Use bactericidal wipes to disinfect roller.


Treatment should be soft and comfortable. Client can fell only a slight tingling or cooling sensation. A tingling sensation can be also perceived under the contact electrode (make sure it is covered by a wet gauze).

Recommended intensity settings for Cata mode (after electrolysis):

- **For Face:** 0.30 mA to 0,80 mA
- **For Body:** 0,80 mA to 1,50 mA

# 05. System Screen Configuration

Configuration screen is accessible by pressing the “tool” icon.

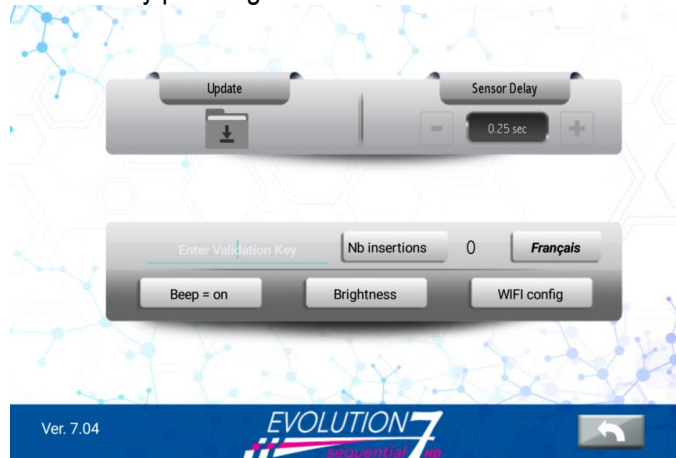

All operating parameters can be modified to suit your needs.

## Insertion Delay (automatic current activation mode)



The **Sensor Delay** key is used to work in the automatic current activation mode, without the pedal. The Sensor Delay is the interval of time between the beginning of the insertion in the follicle and the activation of current passage. This delay can be modified to suit the technician's dexterity and difficulty encountered at the treated area. It may be set from 0.5 sec to 5 sec., in 0.1s increments. It contributes to the technician's comfort and reduce lengthy work fatigue. Using the Sensor Delay necessitates a certain amount of dexterity. The probe must always be accurately inserted into the follicle at the moment of current activation.

### Important note:


- It is not recommended to work with Sensor delay when an area presents difficult insertions or tight follicles.
- If you are not familiar or comfortable with the Insertion delay technique, set time to 2.5 seconds or longer.

## Insertion counter



Registers the number of times the current is actually activated during a specific treatment. Pressing this key for more than one second to reset the insertion counter.


## Sound

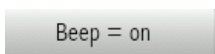

key.

At the end of each insertion, the unit beeps to indicate it is safe to remove the needle from the follicle. If you use the visual satellite signal, silence the beep by pressing this

## Screen brightness

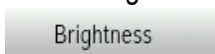


This key is used to adjust screen the brightness which is set at maximum by default. To lower brightness, press this key repeatedly until the desired brightness is reached. Brightness will loop around from high to low when the highest setting is reached.

## Wi-Fi configuration




This key is used to access the Wi-Fi parameters

- First, make sure that the Wi-Fi port is enabled by checking the box next to Wi-Fi.
- When done, the unit will list all available Wi-Fi Networks within range.
- In this example, available Wi-Fi is a secure network (this is noticeable by the lock next to the signal icon)

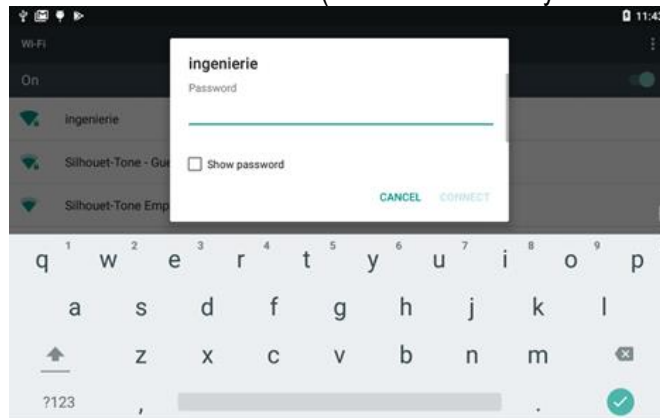

Your Wi-Fi should show up on this screen. If not, your router might not be within range of the unit, is not turned on or not broadcasting its SSID.

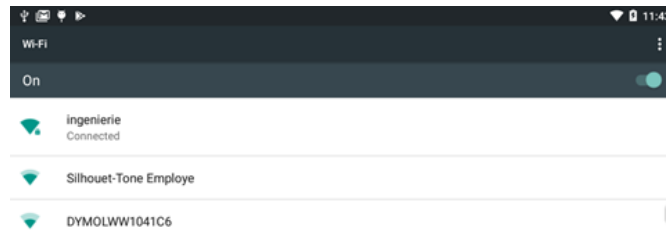

By pressing on the desired network, the unit will try to connect to it. If a password is required, press on the password prompt box to bring up the keyboard and enter password. Consult your network specialist if needed.

When the unit is connected to the internet, the word **CONNECTED** appears under Wi-Fi at the top of the screen.

Turn OFF the unit then turn it back ON by pressing the ON/OFF button on the back; this will register Wi-Fi parameters. Wi-Fi parameters need to be set only once, they are kept in memory until you to change them.



The **Update** key is used to update your system as updates become available. Updates can only be downloaded if your unit is connected to the internet via Wi-Fi.

NOTE: if you have difficulties to connect contact your local distributor or Silhouet-Tone.

## 06. Consultation before Treatments

During consultation, treatment procedures should be clearly explained. Client should know how the treatment will feel and what results she/he can expect.

The number of treatments required will vary between individuals and the areas being treated.

Several factors including hormonal activity, inherited conditions and other influences may impact the effectiveness of treatments.



**IMPORTANT:**

- ✓ Always consult the Contraindications list.
- ✓ Do not treat moles.
- ✓ For all medical procedures refer client to a physician. In case of health problems or medical conditions, or any doubt refer client to a physician or ask for a written authorization from the client's physician.
- ✓ Before starting treatment always have the client sign a "Treatment Release Form".

### Skin Analysis

It is important to keep a client record card to register the client's medical background and medications. (Some medications will promote hair growth.)

### Dehydrated Skin


A dehydrated skin lacks moisture (water) and is **less conductive**.

### Dry Skin

- A dry skin lacks oil (sebum) and will **take longer to heal**.
- Dry skin will be more prone to scabbing\*.
- Instruct client to use a good day and night cream intended for dry skin.

## 07. Hair Growth Cycles

Hair growth cycles play a major role in your hair removal treatment regimen. Excessive hair growth can be caused by a variety of factors including heredity, stress, medication, and hormonal changes. For women, puberty, pregnancy and menopause play important roles in determining hair growth. Researchers estimate there can be up to 2500 hair follicles per square inch but less than 100 hairs will actually be visible at any given time. For these reasons, it is extremely difficult during treatment to predict the exact length of time required before there is no further growth in the treatment area.

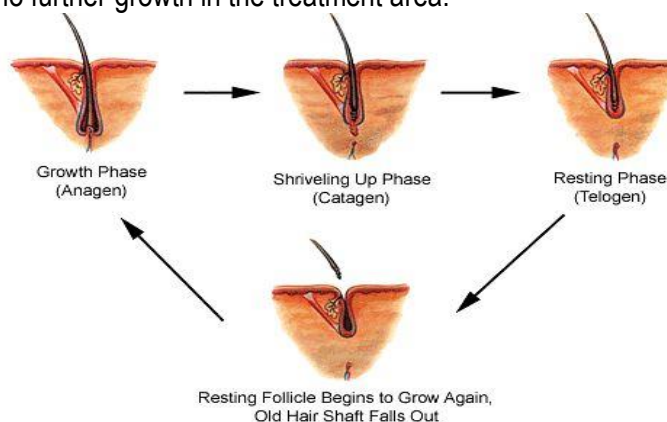

**Anagen:** This is the optimum stage for treatment. The hair is visible above the skin and can be treated. There is plenty of water and salt available in the root area for either Thermolysis or Blend. The hair shaft is attached to the papilla making it easy to access and destroy hair growing cells. This Anagen period for facial and body hair can last up to three weeks.

**Catagen:** The optimum stage for treatment has passed. The growth process has reversed itself and the hair is preparing to shed. The moisture, water and salt content is diminishing. Follicle destruction is almost impossible (hairs can be destroyed in early catagen if the needle is inserted into the follicle to the depth of the root). A follicle treated in the catagen stage will still produce another hair. This stage can last as little as two days.

**Telogen:** This is the final stage, the "resting" stage. The hair is neither developing nor growing. It cannot be treated during this stage, as it usually is not visible above the skin. The hair shaft is fully detached from the papilla, therefore not allowing destruction of the papilla. This phase can be a few days or as long as a few years.

***The "perfect" time for hair to be permanently destroyed is when it is in the Anagen or growing stage.***

It can take 6 to 12 weeks for a hair to grow from the matrix or papilla, to the surface of the skin. Therefore, if you tweeze a hair today, it is generally impossible for you to see the exact same hair in a week. These cycles can vary from individual to individual, and produce patterns that are vastly different.

## 08. Probes & other supplies



### **IMPORTANT:**

- ✓ Use **ONLY** disposable sterile probes and non-insulated
- ✓ Probes have to be discarded after each use.
- ✓ This unit is conceived to use **ONLY** with probes and probe-holders recommended by Silhouet-Tone.

**Probe size varies according to the diameter of the hair to be treated.**

### **Supplies needed for treatment:**

- Pre-depilation cleanser (such as Sensibilis Pre-depilation lotion).
- Gloves (some clients are allergic to latex or vinyl).
- Surgical mask (optional).
- Gauze.
- Soothing gel (such as Sensibilis Post-depilation gel).
- Sun block.
- Container for disposable needles.
- **5 diopters** magnifying lamp.
- Tweezers

## 09. Client "Post Care Instructions"

**After all Electrolysis procedures, the subsequent steps must be followed.**

- Do not touch, stretch or manipulate treated area for at least 24 hours following treatment.
- Clean treated area very gently using fingers and facial cleanser. Do not use face cloth, facial brush or sponge.
- If you wish to wear makeup over treated area, apply foundation very gently, using clean fingers.
- To avoid hyperpigmentation, use a sun block and avoid sun exposure, including tanning salons for one week.
- Apply cold compresses if area feels hot. Do not use ice.
- Apply a medicated preparation such as Polysporin® and the product recommended by your technician after the treatment.
- Refrain from strenuous physical activity for 48 hours after intervention.
- Refrain from using a sauna, whirlpool, or swimming pool for 48 hours following treatment.

- To avoid hyperpigmentation, use sunblock and avoid sun exposure, including tanning salons, for one week. Wear a brimmed hat for outdoor activity until healing is complete.
- Do not drink alcohol 24 hours after intervention.
- Do not take aspirin for 48 hours, after intervention.
- Refrain to use irritating agents (AHA, Retinol, etc.) one week prior and 1 week after electrolysis treatment.

**⚠ CAUTION:**

- ✓ **Never remove existing scabs!** Removing scabs may leave red marks on the skin lasting several months.
- ✓ **ALWAYS give a copy of this instruction sheet to the client.**

## 10. Accessories for Evolution 7HD

Accessories
Footswitch MT-91-5
Contact Electrode
Black Extension Cable
Red Extension Cable
Soothing roller
Screen pen (Sylver)
Clip for needle holder
Needle –holder Short with cable VR20
Needle holder long with cable VR20
Needles
Satellite 2 LEDs for Evolution 7 HD
Quick installation manual ENG
1.6 A Fuse SB (110 / 230V)
Power cord 110V / 220V

- Please verify that you have received all accessories upon reception.
- When ordering, please use the corresponding product codes.
- Please keep box & shipping material to ensure safe shipping when returning your unit for servicing.

**NOTE: please contact Silhouet-Tone if you want to use other probes, probe-holders or any other accessory.**

## 11. Important

- This system makes no medical claims. This procedure is not used in any mean of treating any medical pathological condition. It is intended and should be used by licensed estheticians for esthetic purposes only.
- The masculine generic refers to both the masculine and feminine genders and is used for conciseness purposes only.
- If you have questions regarding the operation of this system, call Silhouet-Tone or your local distributor.
- Silhouet-Tone is not responsible for damage or accidents caused by the incorrect or unprofessional usage of the system.

# 12. Warranty

The Evolution 7HD unit is warranted to be free from any manufacturing or assembly defects for two years from the date of purchase, for parts and labor only.

- Accessories such as the probe holder, cables and contact electrode are not covered by this warranty.
- This warranty does not cover damage caused by accidents, incorrect use, or a party other than SILHOUET-TONE attempting to repair this unit.



**IMPORTANT: Please refer to Silhouet-Tone's Warranty Politics.**

# 13. Specifications

Power supply :	110-240V / 50-60Hz – 1,7A
Dimensions:	Width: 13 in (330 mm) Height : 9 in (230 mm) Depth : 13 in (330 mm)
Weight:	8.8 pounds (4 kg)
Construction:	Thermoformed plastic casing
Electronics:	Microprocessor controlled software
Ambient operating temperature:	+10 to +35 °C
Ambient operating humidity:	30% to 65 % RH

**The unit is CSA and FCC certified**

**SILHOUET-TONE RESERVES THE RIGHT TO CHANGE THESE SPECIFICATIONS WITHOUT NOTICE.**

All rights reserved. No part of this work covered by the copyright hereon may be reproduced or used in any form or by any means– graphic, electronic, or mechanical, including photocopying, recording, taping, or information storage and retrieval systems– without permission of the company.

© Silhouet-Tone Corporation 2021